

Emgality comes in a ready-to-use auto-injector



For patients prescribed Emgality

Do not inject until after you have been shown how to inject Emgality by your healthcare professional

Three simple steps to use Emgality



1. Uncap the auto-injector

Twist off the cap and throw it away. Do not put the base cap back on, as this could damage the needle. Do not touch the needle.



- » Your pen is heat sensitive, remember not to leave the pen out of the fridge for more than 7 days.
- » Be careful not to drop the pen during handling.



2. Place and unlock

Place and hold the clear base flat against the injection site then twist the top to unlock the auto-injector.

- » Be careful not to press the teal button until after you remove the cap, otherwise you may damage the needle.



3. Press and hold, look and listen

Press and hold the teal injection button. You will hear a loud click. Keep holding the clear base firmly against your skin. About 10 seconds after the first loud click, you will hear a second click. Check to see if the plunger is visible. This means your injection is complete.



- » Make sure not to press the teal button until the base of the pen is flat against your skin and you are ready to inject.
- » Wait until you hear the second click before removing the pen from your skin.

Injection sites

- » Remember to change your injection site each month.
- » Your doctor may advise you select from the injection sites in this diagram.



abdomen



buttock



back of arm



thigh

View the **'Directions for Use'** video at mylilly.com.au for easy step-by-step instructions on how to use your Emgality auto-injector.

Ask your doctor for an access code.

Reporting side effects and product complaints

Please review the Consumer Medicine Information (CMI) before taking Emgality, available at www.lilly.com.au/en/products. Adverse events (side effects) and product complaints should be reported. If you are worried about a side effect while taking a Lilly medicine, please contact your doctor or other healthcare professional. To report an adverse event or a product complaint about a Lilly medicine, please call Lilly on: 1800 454 559.

This information is brought to you by Lilly, the manufacturers of this medicine.



EMGALITY® is a registered trademark of Eli Lilly and Company.
Eli Lilly Australia Pty Ltd, 112 Wharf Road, West Ryde NSW 2114.
ABN 39 000 233 992. PP-GZ-AU-0261. Date of preparation: September 2021. EMG0079.

Emgality[®]
(galcanezumab) injection